

# Vision Enhancing Systems®

# VES® USER GUIDE

## Handling, Fitting and Focusing

1. Check to be certain that all screws in the temple hinges and mounting brackets are tight. **DO NOT WEAR** a loose system. Wait until it has been properly adjusted and tightened by your prescribing specialist.
2. Check that the fit of the temples, nose pads, and telescope are proper. If your VES is out of adjustment seek assistance from your prescribing specialist.
3. The frame should fit slightly low on the bridge of your nose to properly position the eyepiece for viewing through the telescope. The Telescope need not be slid down the nose to sight through the eye piece. Simply drop the chin down slightly and look up into the eyepiece.
4. The VES should be handled with two hands and placing it on and removing it from the head very carefully.
5. The VES is front heavy and may slip down the nose. Use of a sport strap to support the VES on the head can be helpful.
6. The VES should not be shared with others. Use by others may place the system out of position or adjustment or may cause damage not covered by the warranty.
7. The VES is focusable for varied distances. The focusing knob should **NEVER** be forced beyond its free range of movement. If the focus becomes jammed, **DO NOT FORCE** it. Return your VES for maintenance and repair to your prescribing specialist.

## Cleaning Your VES

The eyepiece lens may become soiled with dirt or skin oil. This may reduce clarity through the telescope. Clean the front and the rear lenses of the telescope with an approved camera or eyeglass lens cloth.

Moisten only the eyeglass lenses (not the telescope lenses) with water or eyeglass cleaner if required. NEVER hold the VES under running water. The lenses may be blown dry with compressed air to remove dust (available through camera and office supply stores).

## Support or Questions

Call your prescribing eye care specialist or Ocutech for assistance.

**109 Conner Dr. • Suite # 2105**

**Chapel Hill, NC 27514 USA**

**800-326-6460**

**919-967-6460**

**Fax: 919-967-8146**

**info@ocutech.com**

**www.ocutech.com**

## Safe Use and Operation

- ▶ The VES can be used while stationary or while moving. The VES is intended to be used as an aide to the wearer's existing vision through the eyeglass lenses.
- ▶ The VES is not intended to replace the user's "regular vision." The VES will pose the greatest risk to the wearer when it is used while moving or while engaged in any physical activity. The VES should **NOT** be sighted through, or should be removed, when the activity with the VES increases the risk of danger or harm to the wearer.

### **DO NOT continually look through the VES while:**

- Walking, traveling or while in unfamiliar areas
- Coursing stairs, curbs, doorways, escalators, revolving doors, etc.
- Engaged in operating power equipment or other potentially dangerous machinery
- Never look directly at the sun with or without the VES
- The VES will obscure some field of vision above and around the line of sight. Care should be taken to avoid objects such as a low tree branch or car door

## LIMITED WARRANTY Vision Enhancing Systems®

The VES II, VES K, VES Sport, and VES Mini are provided with a limited 2-year warranty against defects in manufacture. Contact your prescribing specialist for service.


### **The Warranty DOES NOT apply In the following cases:**

- Damage due to misuse, mishandling, disassembly, tampering, accident, impact, or dropping.
- Exposure to sand, moisture, excessive heat, chemicals or failure to follow handling and maintenance instructions.
- Damage caused by service by other than a certified Ocutech® repair facility.

- Damage caused by inappropriate use or adaptation of the device.
- Any implied warranties for this product, including warranties of fitness for a particular use or purpose, are limited to the original purchaser, and no warranties either express or implied will apply.
- No other warranty, except as set forth above, given by any person, form or corporation with respect to this product shall bind Ocutech, Inc.
- In no event will the manufacturer of the VES® be liable for any damages, expenses or lost profits, savings, or other damages arising out of the use or inability to use this product.

**OCUTECH®** Vision Enhancing Systems

109 Conner Dr. • Suite # 2105 • Chapel Hill, NC 27514 USA  
800-326-6460 / 919-967-6460 • Fax: 919-967-8146 • info@ocutech.com • www.ocutech.com  
VES® and Ocutech® are Trademarks of Ocutech, Inc.

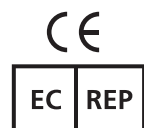

Obelis s.a  
Boulevard Général Wahis 53  
1030 Brussels, BELGIUM  
Tel: +(32) 2. 732.59.54  
Fax: +(32) 2.732.60.03  
E Mail : mail@obelis.net